


# RADIALUX™


## CORDLESS LIGHTED RETRACTOR

### Product features

- Disposable, single-use retractor kit
- Multiple blade configurations for enhanced tissue pocket access
- Total Internal Reflector Lens disperses light for improved visualisation in deep surgical pockets
- No capital required
- Non-conductive

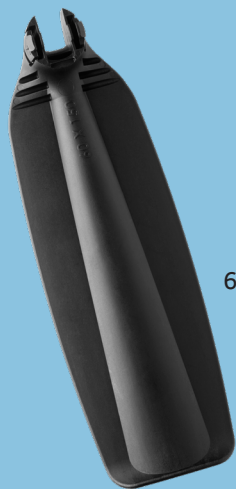

60 x 150mm

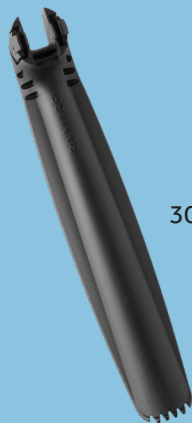

30 x 135mm



60 x 90mm



20 x 50mm

Working through small incisions presents visualisation challenges — especially in deep surgical pockets. The RadiaLux Lighted Retractor is a single-use, cordless retractor specifically designed for deep cavity illumination. Powered by a low temperature LED, RadiaLux features four interchangeable blades developed for fast exchange intra-operatively. With a specially designed lens, light is directed towards the back of the surgical pocket in a unique radius improving visibility in the plane of dissection.

### Surgical applications

- Breast surgery
- Plastic surgery
- Orthopaedics
- CIED procedures

**Medtronic**

# Assembly instructions



## Attaching blades

1. Grasping the retractor blade and handle as shown, angle the blade onto the distal end of the handle.
2. Rotate the blade and insert tabs into the slots of the handle. You will hear an audible click when the tabs have engaged.

**NOTE:** Exercise caution while attaching the blades to avoid skin or glove pinch.



## Detaching blades

1. To switch blade sizes, detach the blade by squeezing the release tabs.
2. Once firmly squeezed, the blade can be rotated out of the handle.

**NOTE:** Do not apply pressure to blade while squeezing release tabs, as this will increase the force required to disengage tabs.

## Technique tips

- Deactivate light when not in use to preserve battery life.
- Eschar buildup on teeth can be removed manually with gloved fingers or gauze pads.

### Ordering information

Description	Catalogue number
RadiaLux	50-101-1

Rx only. Refer to product instruction manual/package insert for instructions, warnings, precautions and contraindications.

## Medtronic

Medtronic Australasia Pty Ltd  
2 Alma Road  
Macquarie Park, NSW 2113  
Australia  
Tel: +61 2 9857 9000  
Fax: +61 2 9889 5167  
Toll Free: 1800 668 670

Medtronic New Zealand Limited  
Level 3 – Building 5,  
Central Park Corporate Centre  
666 Great South Road  
Penrose, Auckland 1051  
New Zealand  
Fax: +64 9 918 3742  
Toll Free: 0800 377 807

Medtronic New Zealand Limited via  
SCIONZ Authorised Sales Agent  
Unit A, 31 Cheshire Street,  
Parnell Auckland  
New Zealand  
Toll Free: 0800 724 669

[medtronic.com.au](http://medtronic.com.au)  
[medtronic.co.nz](http://medtronic.co.nz)

© 2020 Medtronic. All Rights Reserved. ANZ8251-082020